



19 CLINICAL EVALUATORS

56 TOTAL USES

92% CLINICAL RATING

Key features: In-office whitening system • 38% Hydrogen Peroxide
• Dual-barrel syringe dispenser with brush-tip applicator

Description

Pola Rapid is an advanced in-office tooth whitening system:

- Whitens teeth with an incredibly fast 24-minute application time.
- Features built-in desensitizers and fluoride.
- Uses a 38% hydrogen peroxide formulation.
- Simple and precise application.

Indication

- In-office teeth whitening.

Unique Attributes

- Blue gel makes for easy application and visibility.
- Faster system - this material only requires 24 minutes of treatment, significantly reducing patient time in the chair.
- The brush tip allows you to place very easily.
- Light is not necessary; however, the **Radii Xpert** light can be used with the whitening attachment and the **Pola Stand**.



Unique application tip



BEFORE




AFTER

Photos courtesy of Dr. Ona Erdt



Pola Stand with Radii Xpert Light

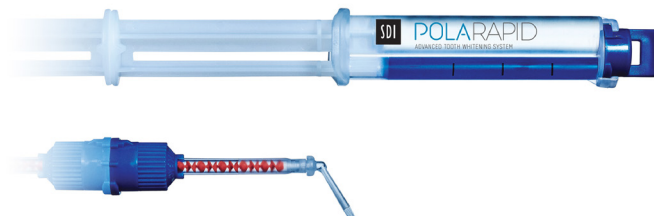

Clinical Tips

- Use a surgical suction to remove the whitening gel between applications for fast and precise removal.
- Double check the barrier between each application and make sure there are no areas of leakage.
- Make the time to take before and after photos. It really shows the patient the difference. Even I did not realize there was such a big difference for a couple of patients until I looked at the photos later.

"GOOD RESULTS, EASY TO USE, AND NO LIGHT NEEDED."

Evaluators' Comments

"Instructions were easy to follow and time for the procedure was excellent."

"I really liked the brush tip and simplicity of the packaging."

"Easy application."

"The quick chair time is amazing. I had patients who have had other brand name in-office whitening comment they this was so much faster and they got better results."

"Color of the gel allowed easy visibility in placing."

"I feel that it worked well both with and without the Radii Xpert light."

"The brush applicator takes a little getting used to. It makes the applicator tip overall wider than you may be used to."

"Worked nicely and didn't drive sensitivity on root exposure patient - able to control application."

"When dispensing the first amount, it was difficult to get the mixing of material just right - seems like a lot wasted."

Consultants who would:

72% Recommend to a colleague


Consultants who would want to stock in their office:

33% Yes, instead of current product

28% Yes, in addition to current product

28% I might want to order this product for certain cases

Evaluation Summary:

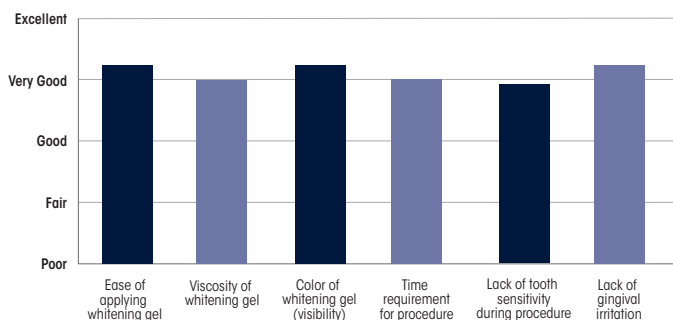

Compared to Competitive Products:

