

**SDI**

# RIVA LIGHT CURE/HV

**1**

Isolate tooth, prepare cavity.  
Apply Riva Conditioner for  
10 seconds, wash and remove  
excess water.

---

**2**


Wash thoroughly.

---

**3**

Remove excess water.  
Keep moist.

---

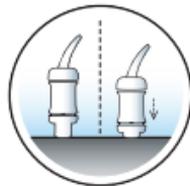**4**


Activate the capsule by  
pressing down. Only activate  
prior to mixing.

---



Immediately mix in a  
triturator for 10 seconds.

---

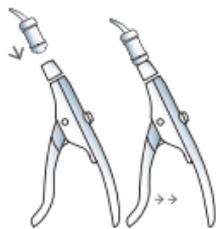**5**

**Immediately** place into capsule  
applicator and click trigger until  
paste is seen through the  
nozzle.

---

**6**

Extrude Riva Light Cure/HV in  
increments of no more than  
2mm into cavity.

---

**7**

Light cure each increment for  
20 seconds.

---



**8**

Final finishing under water spray  
can begin immediately after  
light curing.

---



1

Rounded internal angles and horizontal gingival wall.

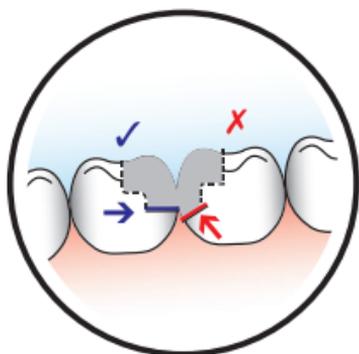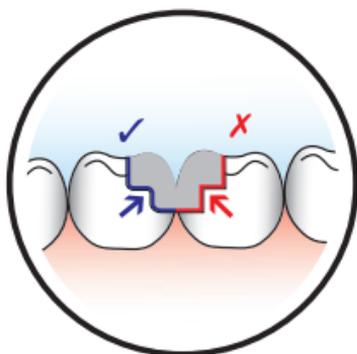

2

No over contours.

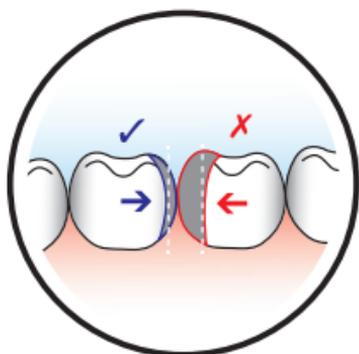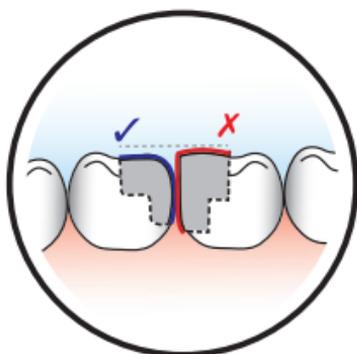

3

Rounded external contour, from 1 to 1.5mm away from cusp peaks.

